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### Tennis Elbow Brace E2

Model: 41301 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

#### Product overview

1)Product name: Tennis Elbow Brace E2

2)Product model: 41301

3)Product specification: F

4)Product Structure:

The product mainly consists of a soft body, a strap, a strap buckle and a magic tape.

5)Operating principle:

The product is designed with light weight and is convenient to use and comfortable to wear. And provides protection for the elbow at the time of high-strength motion.

6)Scope of application:

It is suitable for relieving and preventing pain associated with tendonitis, tennis elbow, repeated stress injury (RSI), and cumulative trauma disorder (CTDS) in a workplace or an entertainment environment.

#### Product features and performance description


1) Product features and storage conditions

**【Product features】** The product is adjustable in size, common in the left and right, and comfortable to wear.

**【Storage conditions】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2)Using the operating

**【Application Instructions】**

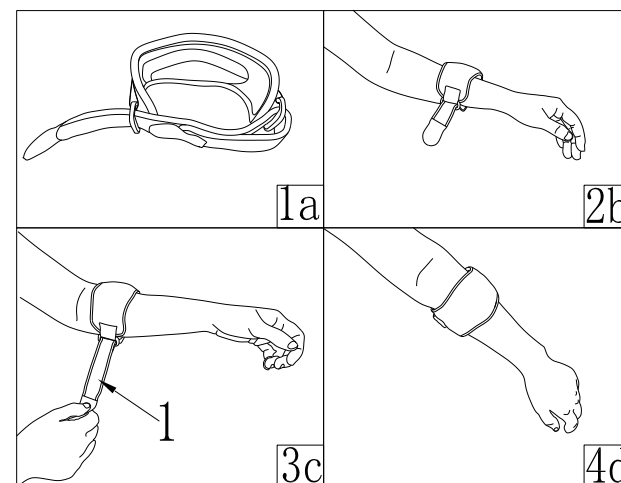#### APPLICATION INSTRUCTIONS

STEP 1: Open the product (Fig1a).

STEP 2: Put the tennis elbow brace in the hand (Fig 2b).

STEP 3: Move the guard to the small arm's muscle and place the compression sponge at the hand arm and tie the strap 1 (Fig3c).

STEP 4: Adjust the tightness of the strap and the position of the compressed sponge (Fig 4d)

**WARNING: The strap should not be too tight, otherwise there will be a risk of affecting the blood circulation. Please carefully adjust the tightness of the strap.**

#### Maintenance:

Product can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

#### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

\* For a single patient only, not recommended for multiple patients.

#### Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

#### Product material

1) Main body: sponge/nylon.

2) Straps: sponge/nylon.

3) Buckle: plastic.

#### Contraindication

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

#### Storage, transportation and expiry date of products

1 )Product is valid: Three years

2 )Product storage: Please place it in a dry and clean room before use to ensure ventilation

3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

#### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

#### Symbol and picture description

 Manufacture Date

 Batch Code

 medical device

 CE Symbol

 Protect from sunlight

 Hand wash only

 Do not use when the packing is damaged

 Warnings and Precautions

 Reference instructions

#### Accessories list


Instructions----- 1 copy

#### After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

#### Simple Medical (Shenzhen) Limited

Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street,Longhua District,  
Shenzhen, China.518109  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: info@simplemedicla.cn`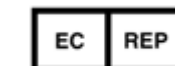

SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

**\*Please refer to the Instructions before usage.**