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Arm Sling E3

(Model: 31302 Size: F)

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

1)Product name: Arm Sling E3

2)Product model: 31302

3)Product specification: F

4) Product Structure: The product is mainly composed of air-permeable mesh cloth, shoulder belt, shoulder pad, chest band, plastic button and magic tape.

5) Operating principle

The product adopts open design and soft breathable material, which makes the product easy to use and comfortable to wear. The reinforced chest band makes the arm difficult to shake and has a double protective effect. The built-in thumb ring can balance the key points, maintain the support strength of the arm, reduce the thumb pressure and help the recovery of the arm.

6) Scope of application: It is used for fracture fixation of arm and upper arm, instability and dislocated fixation of shoulder joint, post-operation and plaster stage, and dislocated wrist joint.


Product features and performance description

1) Product features and storage conditions

【Product features】 The product is Comfortable, breathable, Easy fit, convenient adjustment, Supportive with built-in thumb loops, Reversible on left or right arm.

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating **【Application Instructions】**

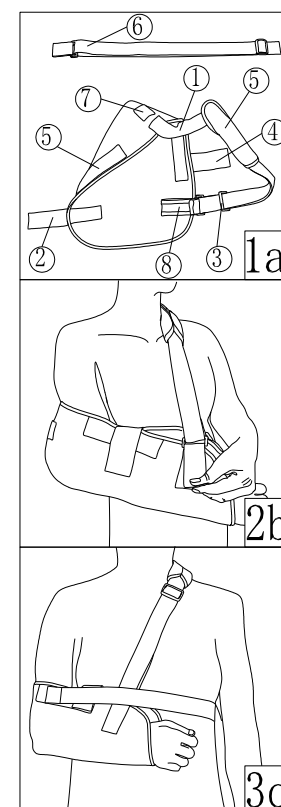STEP 1:open the product (Fig 1a)

STEP 2: The shoulder strap # 1 is diagonally cross over the shoulder, the arm is placed in the net pocket, and the thumb can be hung on the thumb ring # 8. The magic tape # 2 is passed through the plastic ring buckle # 3 and adhered to the wool surface, and the magic tape # 4 is fixed on the surface magic tape # 5. Adjust the length of the shoulder strap # 1 so that the shoulder pad # 5 is in the shoulder comfort position(Fig2b)

STEP 3:First, one side of the chest band is fixed on the surface magic tape # 7, the chest band wraps around the outer side of the body and is adhered to the surface magic paste # 5, the length of the chest band is adjusted, and the injured arm is not easily shaken (Fig. 3c).

Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors

Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

Product material

1)chest strap / shoulder strap: nylon

2) shoulder pad: nylon / sponge

3) big net pocket: nylon

4) buckle: polyethylene

Contraindication

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

1)Product is valid: Three years

2)Product storage: Please place it in a dry and clean room before use to ensure ventilation

3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags) : please dispose of the packaging of this product according to the local laws.

Symbol and picture description

Manufacture Date

Batch Code

medical device

CE Symbol

Hand wash only

Protect from sunlight

Do not use when the packing is damaged

Warnings and Precautions

Reference instructions

Accessories list

Instructions ----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.


Simple Medical (Shenzhen) Limited

Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518109

Tel : +86-755-86038563

Web: www.simplemedical.cn

E-mail: info@simplemedicla.cn`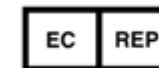

SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

*Please refer to the Instructions before usage