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Multi-functional Arm Sling E4

(Model: 31303 Size: F)

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

1)Product name: Multi-functional Arm Sling E4

2)Product model: 31303

3)Product specification: F

4) Product Structure:

The product is mainly composed of comfortable sponge body, shoulder buckle, chest circumference fastening belt and magic sticker.

5) Operating principle

The product adopts open design and soft and comfortable material, which makes the product easy to use and comfortable to wear. The reinforced chest band makes the arm difficult to shake and has a double protective effect on the recovery of the injured arm.

6) Scope of application:

It is used for Shoulder contusion and strain, Shoulder dislocation, The stability of the top of the humerus fracture, Fracture of the shoulder blade.

Product features and performance description

1)Product features and storage conditions

【Product features】


Open design, easy fit Comfortable, breathable. Firm fixation, reinforced stability.

【Storage conditions】

This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】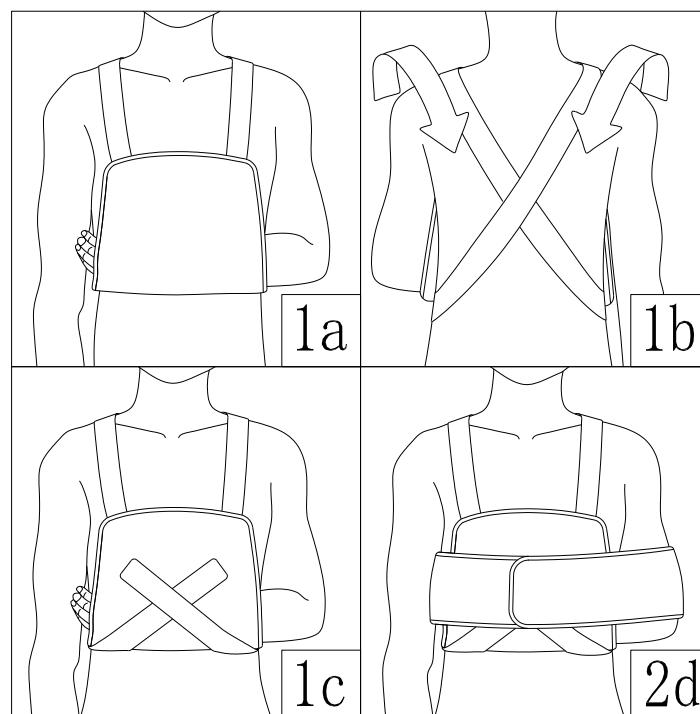

APPLICATION INSTRUCTIONS

STEP 1:Place the injured hand in the sling pocket and place it in front of the chest, and then fasten the shoulder straps around the shoulders from top to bottom in front of the sling pocket (Fig. 1a~1c)

STEP 2: after adjusting the shoulder strap to the appropriate length, fix one end of the chest strap to the surface of the sling pocket, surround the body around the injured arm, and fix the tail end to the chest strap (Fig. 2d).

Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors

Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

Product material

1) shoulder strap: nylon

2) Chest circumference fixed belt : nylon / sponge

3) main body : nylon / sponge

4) buckle: polyethylene

Contraindication

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

1)Product is valid: three years

2)Product storage: Please place it in a dry and clean room before use to ensure ventilation

3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

Manufacture Date

Batch Code

medical device

CE Symbol

Hand wash only

Protect from sunlight

Do not use when the packing is damaged

Warnings and Precautions

Reference instructions

Accessories list

Instructions ----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street,Longhua District,
Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

EC REP
SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage**