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【网址】www.simplemedical.cn

Instructions for Use Clavicle Support E7

(Model: 52301 Size: S/M/L)

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

1)Product name: Clavicle Support E7

2)Product model: 52301

3)Product specification: S/M/L

4)Product Structure:

Products are mainly composed of composite cloth, triangle buckle and magic paste.

5)Operating principle:

The product is designed with light weight and is convenient to use and comfortable to wear. The product is suitable for various purposes, and is convenient to wear and remove.

6)Scope of application:

For external fixation of injuries such as fractures or soft tissues.

Product features and performance description

1) Product features and storage conditions


【Product features】

The product is easy to use, comfortable, light and adjustable.

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2)Using the operating

【Application Instructions】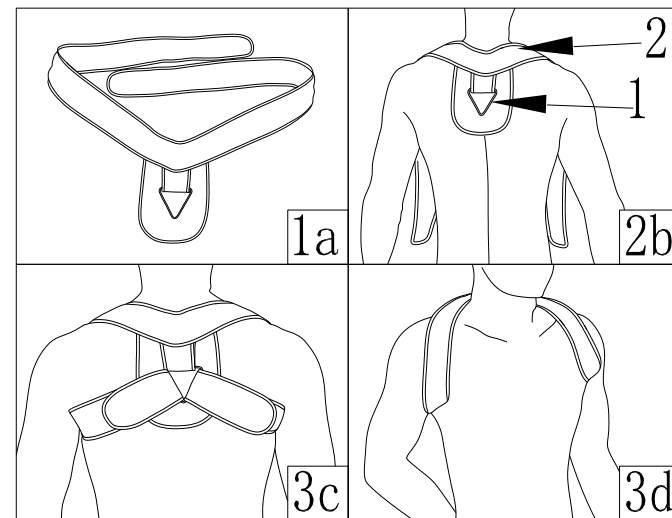

APPLICATION INSTRUCTIONS

STEP 1: open the product (Fig 1a).

STEP2: place the retainer 2 on the back of the shoulder, and the metal triangle buckle 1 pair the quasi-spine (Fig 2b).

STEP 3: refasten the fixed strap 2 through the triangular buckle 1 according to your wearing comfort (Fig 3c~3d).

NOTICE: Do not tighten the fixing tape too tightly; otherwise there will be a risk of affecting the blood circulation. Please carefully adjust the tightness of the fixing belt.

Maintenance:

Product can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage .After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

* For a single patient only, not recommended for multiple patients.

Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

Product material

1) composite cloth: EVA foam / nylon

2) triangle buckle: stainless steel

Contraindication

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

1)Product is valid: Three years

2)Product storage: Please place it in a dry and clean room before use to ensure ventilation

3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

Manufacture Date

Batch Code

medical device

CE Symbol

Hand wash only

Protect from sunlight

Do not use when the packing is damaged

Warnings and Precautions

Reference instructions

Accessories list

Instructions----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street, Longhua District,
Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

EC REP
SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage**