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**Instruction For Use
Shoulder Brace E8**

Model: 31501/31502 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: Shoulder Brace E8
- 2)Product model: 31501/31502
- 3)Product specification: F
- 4)Product Structure:

Shoulder straps are mainly composed of shoulder straps, arm straps and straps.

5)Operating principle

The product is convenient to use, comfortable to wear and good in fixing effect. The fixing band adopts a soft and air-permeable material, so that the patient is more comfortable to wear. The adjustable shoulder strap and the arm strap allow the patient to achieve a better rehabilitation effect.

6)Scope of application

External use for dislocated or loose shoulder joint, acute and chronic pain and inflammation, degenerative lesions of shoulder joint, sprain of shoulder joint.

Product features and performance description


1) Product features and storage conditions

【Product features】 The product is simple and can be applied topically to patients with plantar fasciitis and pain

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature


2) Using the operating

【Application Instructions】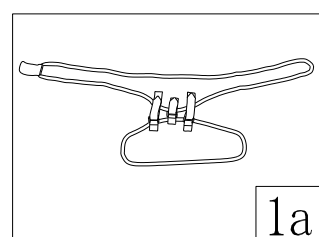


APPLICATION INSTRUCTIONS


STEP 1: open the product (Fig 1a)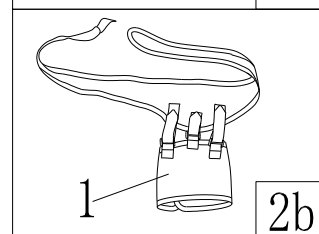

STEP 2: bend the wall band 1 into an arm-sized ring (Fig 2b)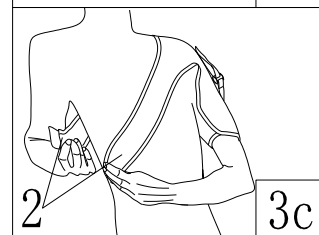

STEP 3: put the shoulder strap on the shoulder through the arm band 1, And Then fasten the shoulder strap 2 back and forth. (Fig3c)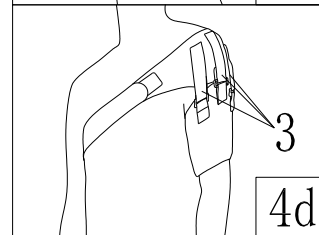

STEP 4:adjust the tightness of arm band 2 and tie band 3 (Fig 4d)

Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Arm strap / shoulder strap: elastic cloth
- 2) Strap: nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

- 1)Product is valid: Three years
- 2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags) : please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol

- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions ----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street,Longhua District, Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage**