



Simple Medical (Shenzhen) Limited
Instruction for Use
Thumb brace W1

Model:31901 Size: F



Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1) product name: Thumb brace W1
- 2) product model: 31901
- 3) Product specification: F
- 4) Product Structure:

The product is mainly composed of a breathable and soft body, supporting device and binding straps.

5) Operating principle

The product adopts lightweight design, easy to use and comfortable to wear The large body and support device with special outline of the product can comfortably provide maximum support for the thumb and limit the movement of the thumb, and the large body with soft fabric can provide comfortable wearing degree for patients Helps relieve pain related to thumb joints and repetitive stress.

6) Scope of application

Mild to moderate thumb joint sprains and strains, postoperative healing and other soft tissue injuries to restore fixed support

Product features and performance description


1) Product features and storage conditions

【Product features】Breathable, comfortable, Light weight, max support, Adjustable straps.

【Storage conditions】This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2)Using the operating

【Application Instructions】

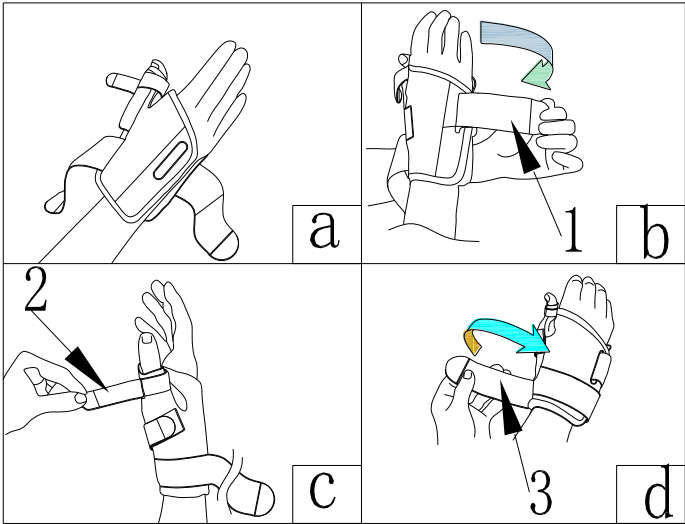

APPLICATION INSTRUCTIONS

- STEP1:** Loosen all the straps put the hands into the holder (fig a)
- STEP 2:**Tie the strap #1 through the lashing hole before tightening it (fig b)
- STEP 3:** Tie the thumb strap #2 through the hole and fasten it (fig c)
- STEP 4:** Tie the strap #3(fig d).

Note: do not tie the band too tight, please adjust the tightness of the band as needed

Maintenance:

Lining can use neutral detergent and 30 ℃ water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

Basic product environment

- 1) Ambient temperature: -15 ~ 60℃
- 2) Relative humidity: 80%

Product material

- 1) Body material: spandex, nylon, polyester PU
- 2) Support sheet: polypropylene
- 3) Strap: nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

- 1)Product is valid: 5 years
- 2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol

- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions ----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address:
519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

EC REP
SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage**