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【网址】www.simplemedical.cn

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: Simple LO 108
- 2)Product model: 52002
- 3)Product specification: F
- 4)Product Structure:

The products are mainly composed of breathable mesh cloth, nylon wire, pull ring, plastic pulley support system and magic sticker.

5)Operating principle:

The product adopts a double independent mechanical pulley system to provide patients with superior compression level, maximum comfort, flexible control, stable support and effective pain relief. The flexible rear pulley system conforms to the patient's body type and can easily adjust the required comfort level when standing or sitting. A single product has many choices of suitable sizes. The contoured filling panel and breathable mesh cloth provide comfortable wearing and smoothness for patients.

6)Scope of application:

It is suitable for Lumbar mechanical back pain, Spinal stenosis, Degenerative disc, Spondylolysis, Spondylolisthesis, Trauma, Herniated, Disc, Bulging Disc, Facet Syndrome

Product features and performance description


1) Product features and storage conditions

【Product features】 The product is Multi-fit sizing options, double independent pulley system, low profile anterior panel, wide adjustments, greater comfort & compliance

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】

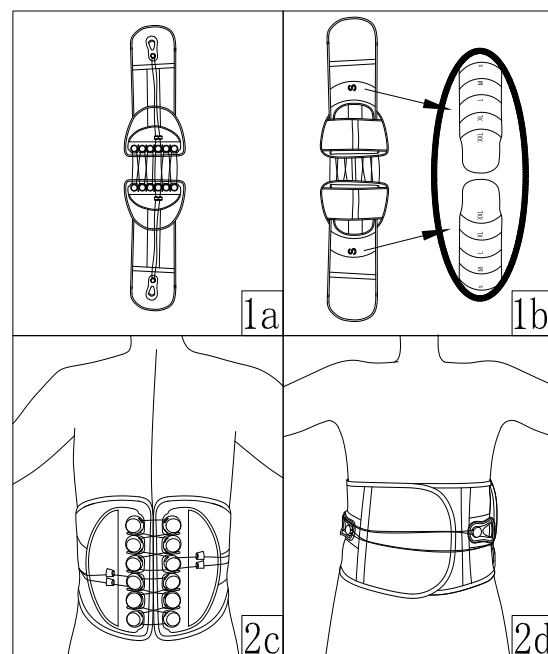APPLICATION INSTRUCTIONS

STEP 1: Open the product, peel off the transverse extension wide fixed band, then pass the wide fixing band through the fixing port, and then adjust to the appropriate size according to the personal body type (inner mark S/M/L/XL/XXL) and fold back into the fixing port according to the edge of the middle connection plate (Fig1a~1b).

STEP 2: Place the Lumbar Brace at the waist, the center is aligned with the spine, and the bottom is at the coccyx. After the fixed belt is fastened to the waist, the hook panel will be affixed to the hairy surface panel. Then pull the pull ring out until the fastening band is tightened and then firmly adhered to the front panel (Fig 2c~2d).

WARNING:

- 1) **The fastening belt is not too tight, please pay attention to the wearing comfort of the fixed belt..**
- 2) **The application instructions listed above are for use only by qualified professionals. Unless otherwise directed by orthopedic professionals, patients are not advised to try to use or adjust the device. Any further adjustment or wear frequency and wear time will be determined by your orthopedic professional.**

Maintenance:

The software part can be washed with a neutral detergent and a 30 & deg; C sailor and air-dried. Do not put the dryer or use other heat sources for drying. Elements such as salt water, sand, dirt, etc. may cause injury. After exposure to these conditions, rinse thoroughly with clear water, dry the towel or air-dry (do not machine the machine). For detergents, use mild detergent and do not use softeners and bleaches. The rigid parts can be wiped clean with a damp cloth.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

* For a single patient only, not recommended for multiple patients.

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Main body: nylon.
- 2) pulley support: polyethylene
- 3) pull ring: thermoplastic elastomer

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

- 1)Product is valid: three years

2)Product storage: Please place it in a dry and clean room before use to ensure ventilation

3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- It is valid for three years
- Protect from sunlight
- Hand wash only
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street, Longhua District,
Shenzhen, China.518131
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: service@simplegroup.net

EC REP
SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

*Please refer to the Instructions before usage.