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Simple LSO 208

Model: 52006    Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: Simple LSO 208
- 2)Product model: 52006
- 3)Product specification: F
- 4)Product Structure:

The product is mainly composed of air-permeable mesh cloth, nylon wire, pull ring, liner, plastic pulley support system and magic tape.

5)Operating principle:

The product adopts a double independent mechanical pulley system to provide patients with superior compression level, maximum comfort, flexible control, stable support and effective pain relief. The flexible rear pulley system conforms to the patient's body type and can easily adjust the required comfort level when standing or sitting. A single product has many choices of suitable sizes. The contoured filler panel and breathable mesh provide comfortable wearability for patients.

6)Scope of application:

It is suitable for Post operative discectomy, IDET procedures, Low back pain/strain, Lumbar disc injury, Compression fractures, Osteoporosis, Disc herniation and degeneration, Spondylolisthesis.

Product features and performance description

1) Product features and storage conditions

【 Product features 】 The product has various size adjustment options, double independent pulley system, profiling filling panel, adjustable pulley rope handle, comfortable and breathable.

【 Storage conditions 】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating


【 Application Instructions 】 **APPLICATION INSTRUCTIONS**

STEP 1: Open the soft gasket on the back (Fig 1a)

STEP 2: Peel the transverse extension wide fixed band through the fixing port and the support port, respectively. Then according to the personal body type, adjust to the appropriate size (inner mark S/M/L/XL/XXL) with the edge of the middle connecting plate as the boundary and fold back into the fixing port and then fix it (Fig 2a~2b).

STEP 3: Close and secure the open back gasket (Fig. 3a)

STEP 4: Place the Lumbar Brace at the waist, the center is aligned with the spine, and the bottom is at the coccyx. After the fixed belt is fastened to the waist, the hook panel will be affixed to the hairy surface panel. Then pull the pull ring out until the fastening band is tightened and then firmly adhered to the front panel (Fig 4a~4d).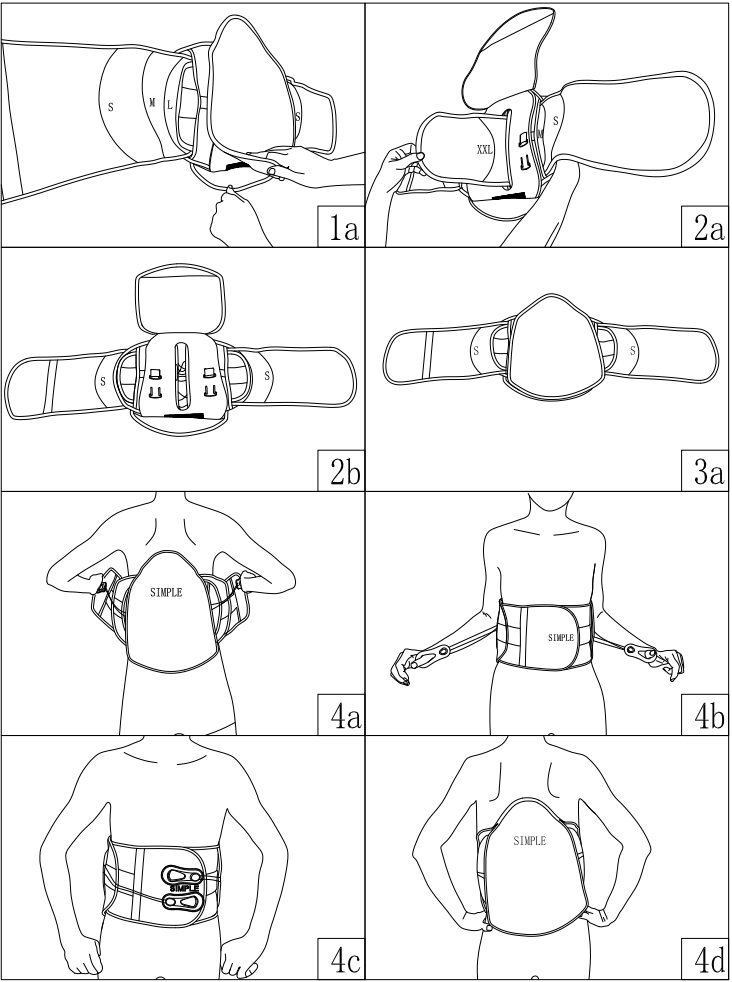

WARNING:

The fastening belt is not too tight, please pay attention to the wearing comfort of the fixed belt.

Proper rehabilitation and activity adjustment are also part of safety treatment.

The application instructions listed above are for use only by qualified professionals. Unless otherwise directed by orthopedic professionals, patients are not advised to try to use or adjust the device. Any further adjustment or wear frequency and wear time will be determined by your orthopedic professional.

Maintenance:

Soft items can be washed with neutral detergents and 30 °C   sailors and dried naturally. Do not put in the clothes dryer or use other heat sources for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with clean water and dry or dry towels (not machine dry). If you need detergent, please use mild detergent: do not use softener and bleach. Rigid parts can be wiped clean with wet cloth.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

**\*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.**

**\* For a single patient only, not recommended for multiple patients.**

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Main body: nylon.
- 2)pulley support: polyethylene
- 3) pull ring: thermoplastic elastomer
- 4)backing strap :nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden.
- ⚠ Those with swollen or ulcerated skin are forbidden

Storage, transportation and expiry date of products

- 1)Product is valid: three years
- 2 )Product storage : Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- 🏭 Manufacture Date
- 🧼 Hand wash only
- ☀️ Protect from sunlight
- 🕒 It is valid for three years
- ⚠ Warnings and Precautions
- 🚫 Do not use when the packing is damaged


Accessories list

Instructions----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street,Longhua District,  
Shenzhen, China.518131  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

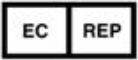SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

**\*Please refer to the Instructions before usage.**