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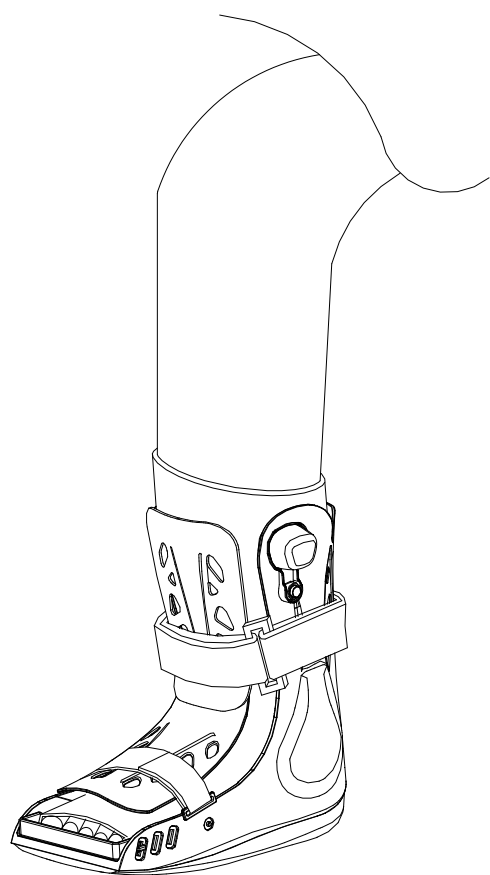
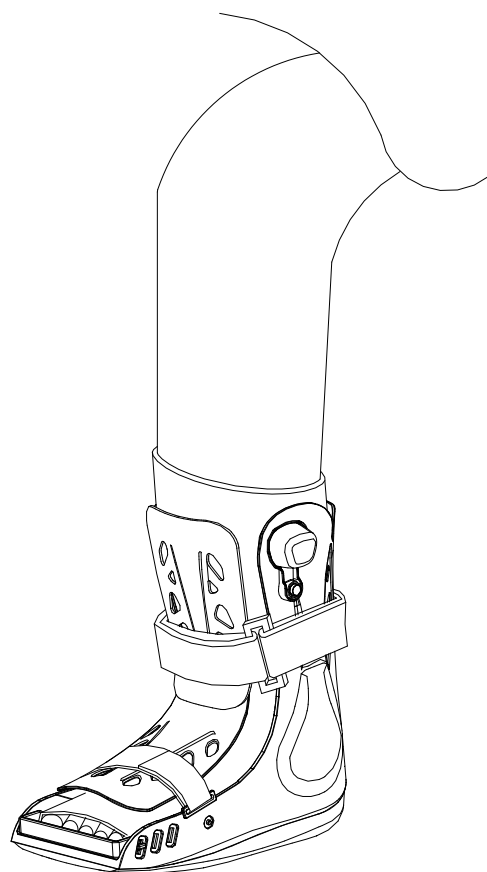
#### Walker B1

Model: 61004      Size: XS, S, M, L, XL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

#### Product overview

- 1) Product name: Walker B1
- 2) Product model: 61004
- 3) Product specification: XS, S, M, L, XL
- 4) **Product Structure:** The product is mainly composed of shell, front cover, strap, lining, adhesive and rivets

#### 5) Operating principle:

The main body of the product is equipped with support, non-skid soft sole and shock absorber. The ankle joint and calf are properly protected by supporting device. Through software and shock absorption systems, patients have more comfort in their legs during exercise. The combination of the upper body of the shoe and the front cover of the calf makes the leg of the patient well protected. The breathable soft insole makes the product provide comprehensive protection to the foot before and after the use of the foot, prevent secondary injury, and provide soft contact between the calf and the fixed support to make it more comfortable to walk. The combination of soft shock absorber and anti-skid soft sole can give the patient's foot a good cushioning effect and help the patient get a good recovery.

6) **Scope of application:** External use is foot fracture, foot ligament injury fixation, foot soles and toes injury fixation, mild and moderate thumb valgus, plantar pain, postoperative recovery and so on clamping bone fixation or support.

#### Product features and performance description

##### 1) Product features and storage conditions

【Product features】 The product is simple, and the self-pressure type double-air bag linkage design is adopted, and is comfortable and air-permeable.

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

##### 2) Using the operating 【Application Instructions】

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**Note1: Wear the sock before wearing the ankle joint! Cotton or gypsum liners will increase comfort-not attached.**

**STEP 1:** Untie the strap, remove the front panel, and open the liner.

**STEP 2:** Sit down and put your feet in the protective gear, with the heel close to the back of the guard. Wrap the lining ring foot with the leg (Fig A).

**STEP 3:** Place the front panel so that the (panel logo) is located above the toe, and fasten the upper panel to the lining, bottom to top to tighten the straps until they feel secure and comfortable (Fig B~C).

**STEP 4:** Adjust the airbag pressure: the initial airbag of the simple walking guard is emptied. Start inflating the airbag from the injured side, place your finger on the top of the inflatable pump of the airbag, and press down until the airbag is fastened. Also inflate the airbag on the other side until the airbag is firm. If inflated too much, please vent the airbag properly. If you want to empty the airbag, hold down the small button that needs to empty the airbag until you feel the right pressure.

**Note 2: 1.** Do not inflate the airbag too much. Inappropriate balloon inflation may cause significant stimulation in patients with diabetic neuropathy. If any visible skin changes occur or the patient reports discomfort, reduce the pressure on the airbag. Do not use this instrument for patients who are unable to express their discomfort.

**2.** When walking on a slippery surface, care should be taken to avoid injury. At high altitude, the expansion of airbags will exceed the optimal level.

**3.** Be sure to check before using it. Do not use damaged products. If there is any tear, wear and tear in this instrument, please stop using it.

#### Maintenance:

Lining can use neutral detergent and 30℃ water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying.

#### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

#### Basic product environment

1) Ambient temperature: -15 ~ 60 ℃

2) Relative humidity: 80%

#### Product material

- 1) shell: polypropylene
- 2) front panel: high density polyethylene
- 3) adjustment belt: nylon
- 4) pad: nylon / polyurethane / polyester (outer net / foam / inner mesh)
- 5) adhesive: polyester
- 6) shoe outsole / shock absorber / shoe inner sole: thermoplastic elastomer / polypropylene
- 7) insole: EVA
- 8) rivets: stainless steel

#### Contraindication

- ⚠ Use with caution for unstable fractures;
  - ⚠ Use with caution in osteoporosis;
  - ⚠ Use with caution for cognitive impairment;
- Use with caution for bony ankylosis;

#### Storage, transportation and expiry date of products

- 1) Product is valid: five years
- 2) Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dust-proof, moisture-proof and pollution-proof during storage and transportation

#### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

#### Symbol and picture description

- Manufacture Date      **LOT** Batch Code
- It is valid for five years      Protect from sunlight
- Hand wash only
- Warnings and Precautions
- Reference instructions

#### Accessories list

Instructions ----- 1 copy

#### After-sales warranty

Warranty: all products from the date of purchase as the original date, normal wear and tear of the product in the use of the process is not considered defective.

The following parts from the date of purchase to the warranty period, if there is any material or process defects, please contact your seller directly, Simple medical will complete or partial repair or replacement of equipment and accessories.  
polymer material shell ---- 1 year

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street, Longhua District,  
Shenzhen, China.518131

Tel : +86-755-86038563      Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

**EC REP**  
SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

\*Please refer to the Instructions before usage.