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SIMPLE Air Heel A1

Model: 62007 Size: S, M, L

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

**Product overview**

- 1)Product name: SIMPLE Air Heel A1
- 2)Product model: 62007
- 3)Product specification: S, M, L
- 4)Product Structure:

The product is mainly composed of breathable soft material, air bag, plastic parts, strap and so on

5)Operating principle

The product adopts lightweight design, easy to use and comfortable to wear. The product adopts self — pressing double airbag linkage design, massage compression, has two supporting devices, and adopts breathable and soft fabric to improve the wearing comfort of the patient. To help relieve the pain of the patient.

6)Scope of application

It is used in patients with Achilles tendonitis, Acute and chronic Plantar fasciitis pain, Heel pain caused by spurs and Heel spurs.

**Product features and performance description**


1) Product features and storage conditions

**【Product features】** The product is simple, and the self-pressure type double-air bag linkage design is adopted, and is comfortable and air-permeable.

**【Storage conditions】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2)Using the operating

**【Application Instructions】**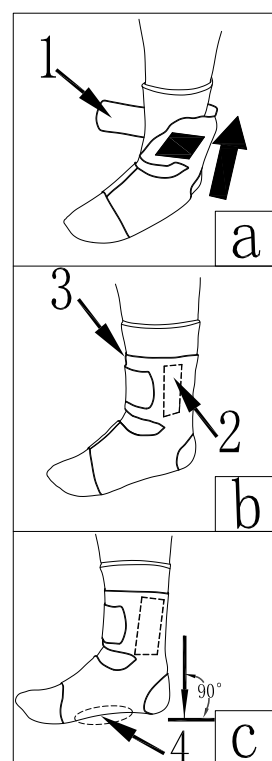

**Note1: please put on socks before wearing Ankle Brace.**

**STEP 1:**Loosen the strap#1, put the foot into the holder facing the bandage ( Fig a)

**STEP 2:**Adjust the airbag #2 and #3 of the retainer so that the airbag is located medial and lateral to the ankle ( Fig b)

**STEP 3:** Stand your feet on the floor, adjust the arch air bag 4 so that it is under the arch, and then tie the strap#1 (Fig c).

**Note 2: the straps should not be too tight. Please adjust the length of the strap as needed**

**Maintenance:**

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

**Warning:**

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

**Basic product environment**

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

**Product material**

- 1) Body material: Spandex fiber
- 2) Support sheet: Polypropylene
- 3) Balloon: TPU
- 4) Strap: nylon

**Contraindication**

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden.
- ⚠ Those with swollen or ulcerated skin are forbidden

**Storage, transportation and expiry date of products**

- 1)Product is valid: five years
- 2 )Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dust-proof, moisture-proof and pollution-proof during storage and transportation.

**Disposal of wastes**

Disposal of wrapping paper (bags) : please dispose of the packaging of this product according to the local laws.

**Symbol and picture description**

- Manufacture Date
- Batch Code
- medical device
- CE Symbol
- Hand wash only

- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- It is valid for five years
- Reference instructions

**Accessories list**

Instructions ----- 1 copy

**After-sales warranty**

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street,Longhua District,  
Shenzhen, China.518131  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

**\*Please refer to the Instructions before usage**

62007 V0.2