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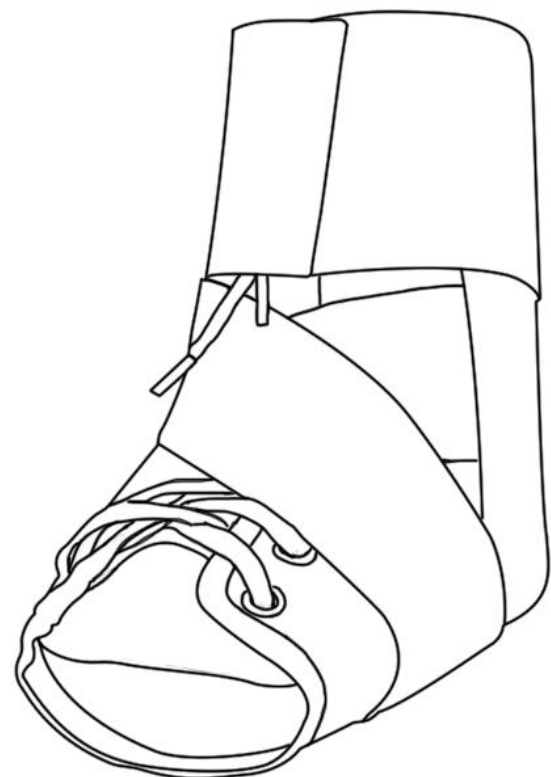
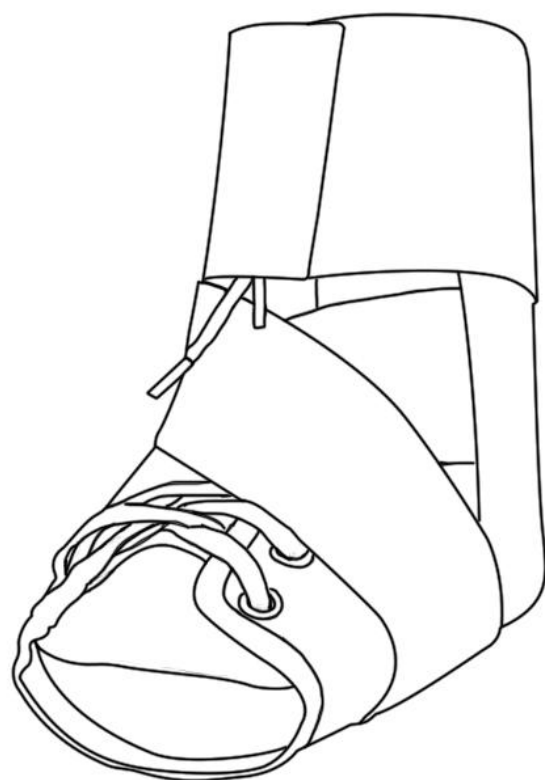
SIMPLE Lace-up Ankle Brace A2

Model: 62005 Size: XS, S, M, L, XL, XXL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference.

Product overview

- 1)Product name: SIMPLE Lace-up Ankle Brace A2
- 2)Product model: 62005
- 3)Product specification: XS, S, M, L, XL, XXL
- 4)Product Structure:

The product is mainly composed of breathable and comfortable body, straps, telescopic belt and so on.

5)Operating principle:

The product adopts lightweight design, easy to use, comfortable to wear and good fixation. The product uses "8" shaped fixing belt and shoe strap, which can effectively fix the injured ankle joint and prevent sports injury. Large body is made of soft, breathable fabric to improve patient's comfort.

6)Scope of application:

Suitable for Mild to moderate ankle distortion, Chronic ankle instability, Prophylactic and therapeutic use in sport activities.

Product features and performance description

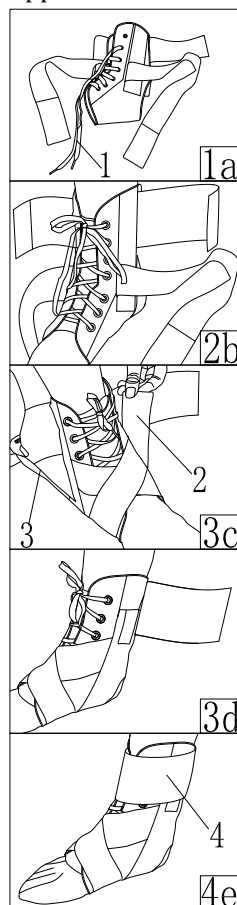
1) Product features and storage conditions

【Product features】 The product is Mesh fabric, great ventilation, Better fitting, supportive, light weight, Lace-up design, custom fastening, Fits right and left foot.

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature.

2)Using the operating

【Application Instructions】

APPLICATION INSTRUCTIONS.

STEP 1: open the product and wear the shoelace # 1 (Fig 1a).

STEP 2: put your feet into the product, adjust the tightness of the straps and tie the straps # 1 (Fig 2b).

STEP 3: straps # 2 / 3 cross around the ankle and fasten firmly (Fig 3c~3d).

STEP 4: adjust the tightness of all straps to ensure that the comfort of wearing brings the straps on both sides closer to the front and fastens them. (Fig 4e)

NOTICE: The straps should not be too tight. Please adjust the length of the strap needed.

Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Body material: Spandex fiber
- 2) Support sheet: plastic
- 3) Straps: nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

- 1) Product is valid: 5 years.
- 2) Product storage: Please place it in a dry and clean room before use to ensure ventilation.
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol
- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged

- Period of Validity 5 years
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions ----- 1 copy

Selection

Item	REF. 62005	
Size	XS, S, M, L, XL, XXL	
	Size	Measurement method
XS	250-280mm	
S	280-300mm	
M	300-330mm	
L	330-350mm	
XL	350-380mm	
XXL	380-400mm	

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518131
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: service@simplegroup.net

SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit 1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

*Please refer to the Instructions before usage

62005 V0.2