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SIMPLE LO 106

Model: 52008 Size: S, M, L, XL, XXL, XXXL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: SIMPLE LO 106
- 2)Product model: 52008
- 3)Product specification: S, M, L, XL, XXL,XXXL
- 4)Product Structure : The products are mainly composed of breathable mesh cloth, nylon wire, pull ring, plastic pulley support system and magic sticker.
- 5)Operating principle:

The product adopts a double-independent mechanical pulley system to provide the patient with superior compression level, maximum comfort, flexible control, stable support and effective relieving pain. The flexible back-pulley system, in line with the patient's body type, can easily adjust the desired comfort in standing or sitting. The low profile front panel and the breathable mesh are comfortable and smooth for the patient.

6)Scope of application:

It is suitable for Lumbar mechanical back pain, Spinal stenosis, Degenerative disc, Spondylolysis, Spondylolisthesis, Trauma, Herniated Disc, Bulging Disc, Facet Syndrome

Product features and performance description

1) Product features and storage conditions

【Product features】 The product is double mechanical pulley system, single front fastener, adjustable pull cord, comfortable, breathable.

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating **【Application Instructions】**

STEP1: open the product (Fig 1a).
STEP2: Place the lumbar retainer in the waist, with the center aligned with the spine and the lower part at the tailbone. Attach the hook face panel to the wool face panel after the fixed belt is attached to the waist. Then pull four Velcro strips outward according to the degree of adaptation until the fixing belt is tightened and then firmly adhered to the front panel (2b~3c).
WARNING:

- 1) The fastening belt is not too tight, Please pay attention to the wearing comfort of the fixed belt.

- 2) The application instructions listed above are for use only by qualified professionals. Unless otherwise directed by orthopedic professionals, patients are not advised to try to use or adjust the device. Any further adjustment or wear frequency and wear time will be determined by your orthopedic professional.

Maintenance:

Product can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry). If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

* For a single patient only, not recommended for multiple patients.

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Main body: Nylon/Stainless steel
- 2) Front panel: thermoplastic elastomer/ Nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

- 1) Product is valid: five years
- 2) Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- medical device

- It is valid for five years
- Protect from sunlight
- Hand wash only
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions----- 1 copy

Selection

Item	REF. 52008	
Size	S, M, L, XL, XXL, XXXL	
Colour	Grey	
Size	Waist size	Measurement method
S	52-62cm	
M	63-74cm	
L	75-89cm	
XL	90-105cm	
XXL	106-120cm	
XXXL	121-140cm	

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
 Address: 519 Wisdom Valley, No.1010 Bulong Road,
 Minzhi Street, Longhua District,
 Shenzhen, China.518131
 Tel : +86-755-86038563
 Web: www.simplemedical.cn
 E-mail: service@simplegroup.net

SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

*Please refer to the Instructions before usage.

52008 V0.2